

# **HEMASEAL & CIDE**

## *Dentin Sealing, Wetting, Desensitizing Agent*

### Directions for Use

Do not store above 77°F (25°C)  
Refrigerate for long storage periods.  
Do not freeze.  
Allow to warm to room temperature before use.

**CAUTION:** If contact occurs with eyes, rinse with water immediately and contact physician.

*Do not use on patients with sensitivity to chlorhexidine.*

### Under Composites

1. After etching, rinse thoroughly with water only. Suction away excess with high volume evacuator.
2. **APPLY ONE OR TWO DROPS HEMASEAL & CIDE INTO DAPPEN DISH. USING MICROBRUSH OR OTHER BRUSH, APPLY GENEROUS COATING AND SCRUB INTO DENTIN FOR 10-15 SECONDS. SUCTION AWAY EXCESS WITH HIGH VOLUME EVACUATOR OR WICK AWAY WITH BRUSH<sup>(1)</sup> TO LEAVE DENTIN LIGHTLY WET<sup>(2)</sup>. APPLY ADHESIVE AND CURE.**  
<sup>(1)</sup> Touch brush to prep, then patient napkin and repeat.  
<sup>(2)</sup> After suctioning or wicking away excess, no pooling of liquid should exist. Dentin should glisten with moist coating.

**FOR UNI-DOSE APPLICATION: CUT TIP AND DISPENSE DIRECTLY ONTO TOOTH OR PLACE IN DAPPEN DISH AS DESCRIBED ABOVE.**

### Under Ionomer or Modified Ionomers – After Crown Completion – Before Final Cementation

1. Clean prep, suction or lightly blow away excess moisture.
2. Apply HEMASEAL & CIDE for 10-15 seconds with microbrush or other brush.
3. Suction or wick away excess HEMASEAL & CIDE and proceed with ionomer placement, temp. or final cementation.

### For Use with Self-Etch & Bond Products (i.e. Clearfil SE Bond, Prompt-L-Pop)

1. Prep tooth.
2. Apply HEMASEAL & CIDE over prep for 10-15 seconds.
3. Wick away excess leaving dentin slightly wet.
4. Apply etch and bond product as directed.

### Under Amalgams

1. Preferred: Rinse prep and blow away syringe water. Apply HEMASEAL & CIDE. Wick away excess. Apply thin layer of ionomer. Place amalgam.
2. If not possible, rinse prep. Blow away excess water. Apply HEMASEAL & CIDE gently. Suction away excess (don't desiccate). Place amalgam.

### Cervical Sensitivity

1. Clean with EDTA for 15 seconds and blot away. Apply HEMASEAL & CIDE for 15-20 seconds. Suction or wick away so lightly wet. Cover with dentin adhesive agent such as Prime & Bond.