

Here is what Opinion Leaders say:



Item# 100: MatrixButtons (Bag of 72)

**Item# 101: MatrixTemp Shade A1/B1:
50ml Cartridge with 10 Mixing Tips**

**Item# 102: MatrixTemp Shade A2:
50ml Cartridge with 10 Mixing Tips**

**Item# 103: MatrixTemp Shade A3:
50ml Cartridge with 10 Mixing Tips**

**Item# 150: Separating Medium Liquid
(0.5 oz.) (Recommended if using Powder/
Liquid acrylics)**



AdvantageDental Products, Inc.

P.O. Box 415 Lake Orion, MI 48361

CALL: 800-388-6319

<http://advantagedentalinc.com/index.html>



"These work well, and while I'm not sure I save a lot of time, compared with the standard PVS technique, I know I save on overall cost. I have sufficient working time, and get a detailed matrix for the provisional. I recommend it."

Gregori Kurtzman, DDS, MAGD, FACD Silver Spring,



"I've been using the buttons for over 10 years. Because they are rigid, I get good margins: quick and easy - to - use. I have plenty of working time, using my everyday water bath. I use these in many of the courses I teach."

Bruce Crispin, DDS, MS Los Angeles, CA



"These buttons are just plain easy. I use them for single crowns and longer spans, with Luxatemp, Prottemp and the MatrixTemp. My assistant just uses a hot cup of microwaved water and we get plenty of working time. With cold water for air-spray, they turn opaque and rigid in seconds and they take up very little space, so I can easily save them, if needed".

Brian Gray, DMD, Washington, DC



"This is a very innovative, accurate and simplistic product/technique that saves me more than 40% in time, with a simple learning curve; a paradigm shift from traditional pre-op matrix impressions."

Frank Milnar, DDS, AAACD St Paul, MN



"A simple and very economical technique. We heat a cup of water with our "quick-hot" tap, and once we got the heating time right, we had plenty of working time, and got a good result with the temporary materials I was already using. They may not save me much time, but they sure save money on materials. I can recommend these for single crowns and longer spans."

Troy Schmedding, DDS, Walnut Creek, CA



"These are so easy to use, with ample working time, and I am able to achieve good margins and occlusion with the provisionals made using this thermoplastic matrix system. I would recommend them to my colleagues."

Ian Shuman, DDS, MAGD, AFAAID Pasadena, MD



"The thermoplastic MatrixButton is a material that can be used easily for single or, by combining multiple buttons, multiple crown and bridge provisionals. This is a low-cost, easy-to-use material that I can simply heat in a hot water bath or a ceramic coffee mug in the microwave while I'm waiting on anesthesia." (excerpted from AGD Impact, March 2018).

Howard S. Glazer, DDS, FAGD Ft Lee, NJ

Matrix™ System

MatrixButtons & MatrixTemp

**The Permanent
Solution to Temporaries**

AdvantageDental Products, Inc.

Custom temporaries have never been quicker....or easier!

The simplest way to make custom impressions.

These products are very fast, easy, accurate and cost-effective. MatrixButtons have no gag, odor or taste issues. The impression is small, stable and can be saved to make a new temporary, in the rare event of fracture or loss of the original. When hardened, the matrix is flexible enough to be removed without distortion, and rigid enough to contain the material and drive deep margins.


You can be in and OUT in 95 SECONDS!

- Simply soften a button (or two) in hot water
- Form it around the tooth or teeth to be prepped
- Blow air to harden
- Then remove. Your impression/matrix is ready!

6 Additional Uses For MatrixButtons:

1. Isolate sensitive teeth while operating in another area
2. Anterior anti-bruxing device to offset clenching & grinding
3. Block out of the space for the pontic
4. Bite registrations
5. Implant shims to guide placement
6. Border molding trays for denture impressions

From This.....to THIS in 95 Seconds!

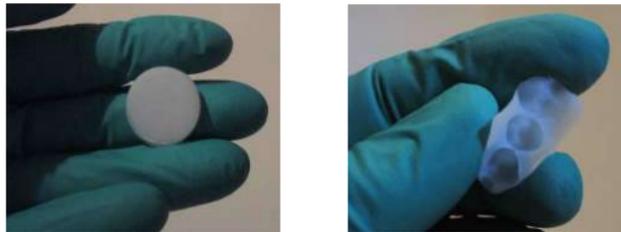

When the MatrixButtons are submerged in hot water- at ~160° ;about the temperature of a hot cup of coffee- they will go from opaque white to transparent, and it is then ready! It takes only a few seconds to position it with the fingers, and seconds more to cool it back to white for removal.

All this takes less than the time VPS requires to cure!

Save Time. Save Materials. Save Money.

The cost of a MatrixButton is \$0.72..... LESS THAN THE PRICE OF 1 VPS MIXING TIP*!

Compare that with the cost of an alginate impression or a Vinyl Polysiloxane impression. PLUS.....

- **NO Mixing!**
- **NO Trays!**
- **NO Clean-up!**

How much is your time worth?



*Microwave oven energy varies.



Perfect for making temps with our MatrixButtons

MatrixTemp is a 1:1 auto-mix product used in the standard "gun". It can also be extruded and hand-mixed and placed. The higher viscosity may require more "grip" than some other brands, but the clinical results are worth it.....Read on!



Additional Advantages of MatrixTemp:

- Superb strength and fracture resistance
- Extended rubber-setting phase allows removal from interproximal undercuts
- Thicker thixotropic consistency allows easy mixing and placement, with or without the mixing tip. No Dripping!
- This viscosity drives **SHARP, CLEAR MARGINS**
- No exothermic heat
- Available in shades A1/B1, A2 and A3



*Luxatemp and Protemp are not products of Advantage Dental Products, Inc. Prices from Henry Schein Catalog, July 1 2019.

Sells for 44% less than Luxatemp® and / or Protemp®*

Clinical Technique



Video Scan me

